

# BIOCREDIT COVID-19 Ag


## SARS-CoV 2 Antigen test

- » The first commercialization Black Gold Particle technology In the world (high sensitivity and specificity)
- » User friendly Dual Color System (Control line: Red, Test line: Black)
- » Rapid Test Time (5~8 Min)
- » Simple detection of SARS-CoV2 antigen

### Core Technology Black Gold Particle



Differentiated Black Gold Conjugate Technology

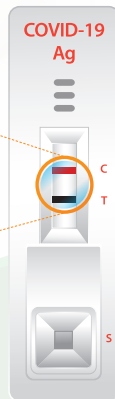

Dual Color System

8 minutes is ok!



YouTube BIOCREDIT COVID-19



## BIOCREDIT COVID-19 Ag rapid test kit

<b>Intended Use</b>	Detection of SARS-CoV2 antigen
<b>Package</b>	20 Tests / kit
<b>Storage</b>	1~40°C
<b>Specimen Type</b>	Nasopharynx / Nasopharyngeal
<b>Shelf life</b>	12 months from manufacture
<b>Time to result</b>	date 5~8 minute

# COVID-19 Rapid Diagnostic Test

## PROCEDURE


Insert the swab specimen and swirl the swab 5-10 times.

Remove the swab while gently squeezing the head of the swab.

Close the assay diluent tube with a filter cap securely.

Invert the assay diluent tube and gently squeeze it to draw 3-4 drops (90~150 $\mu$ l) into a sample well on the device.

Read the result within 5-8 minutes.

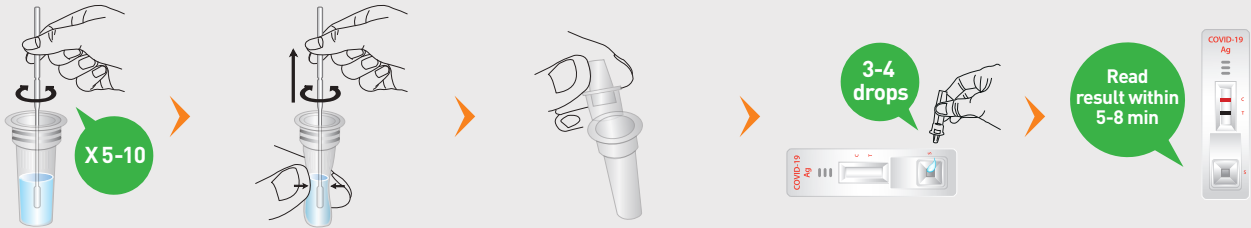

## INTERPRETATION

NEGATIVE




One red line "C" within the result window.

POSITIVE



Two bands, **black** To test line and **red** "C" control line within the result window.

INVALID

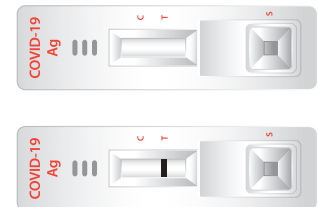

No "C" line within the result window. It is recommended that the specimen be retested.

